carefix[®]

Post-Surgical Bras | حمالات الصدر المخصصة للارتداء بعد العمليات الجراحية | Pooperační podprsenky
Post-operative bh'er | Post-OP-BH | Μετεγχειρητικοί στηθόδεσμοι | Sujetadores postquirúrgicos
Rintaliivit leikkauksen jälkeiseen käyttöön | Soutiens-gorge post-chirurgicaux | Postoperativni grudnjaci
Műtét utáni melltartók | Reggisenipost-chirurgici | 術後用ブラジャー | 수술후보정용 브라 | Postoperatieve beha's
Post-kirurgiske BH | Biustonosze pooperacyjnej | Sutiás pós-operatórios | Послеоперационные бюстгальтеры | Pooperačné podprsenky | Modrčki za uporabo po kirurškem posegu | Post-hirurški grudnjaci | Postoperativa brösthållare | 术后文胸

Instructions for Use | تعليمات الاستعمال | Návod k použití | Bruksanvisning | Gebrauchsanweisung | Οδηγίες χρήσης
Instrucciones de uso | Käyttöohje | Instructions d'utilisation | Upute za uporabu | Használati utasítás | Istruzioni per l'uso
使用説明 | 사용 지침 | Gebruiksaanwijzing | Bruksanvisning | Instrukcja użytkowania | Instruções de uso | Инструкция по использованию | Návod na použitie | Navodila za uporabo | Uputstvo za upotrebu | Bruksanvisning | 使用说明

EN	4
6	AR
CS	8
DA	10
DE	12
EL	14
ES	16
FI	18
FR	20
HR	22
HU	24
IT	26
JA	28
KO	30
NL	32
NO	34
PL	36
PT	38
RU	40
SK	42
SL	44
SR	46
SV	48
CN	50
 1-3 & 5	52-57

Intended use (Post-Op Bras): Fixate and support the breasts and protect wound and scar following breast surgery. Fixate bandages or other medical devices following breast surgery.

Intended use (Comfort Bras): Fixate and support the breasts and protect wound and scar when full movement of arm is regained following breast surgery and during/after radiation therapy.

Intended users: Women, following breast surgery as described under Indications.

Indications

For 24 hour post-surgical use, following the surgeries in ① 1

Clinical benefits

- May reduce the harmful build-up of blood and tissue fluids
- May minimize postoperative swelling (edema) and pain
- May improve blood circulation and wound healing
- May reduce risk of infection
- May protect sensitive and sore scar tissue and reduce inflammation
- May contribute to a nicer scar appearance and an improved surgical outcome
- May provide enhanced and early postoperative mobilization
- May shape and support the operated breast

Compression Level

The Post-Surgical Bras are ranked according to each other in five levels of compression: High (1), Firm (2), Intermediate (3), Moderate (4) and Light (5). Figure shows a compression level of 2 (Firm) and is displayed on the packaging for the specific bra and in ① 2

Compression and Support Zones (Tytex ZoneTech®)

Table 4 – Firmness indicates the different knitted zones within the specific Post-Surgical Bra.

The lower the number, the firmer the knitting; increased compression and support. Compression and support zones for each specific bra are displayed in ① 3

ZONE	FIRMNESS
	1
	2
	3
	4
	5
	6

① 4 – Firmness

- The knitting is unique to each specific Post-Surgical Bra and cannot be compared across the bra range.

Care Instructions

- Find care instructions on the bra
- Close the bra before washing
- Wash with similar colours
- Wash in a laundry bag
- Use of chlorine based products may decrease durability and function

Properly and safe use of the device

- Do not use if packaging is damaged

Additional Instructions

Single patient multiple use

Keep dry

Keep away from direct sunlight

May be disposed with regular household waste

Reporting obligation

If you have any complaints, please contact your supplier directly. In the events of a serious incident that could lead to a significant deterioration of health or to death, contact the manufacturer and the competent authority of the member state. Serious incidents are defined in article 2 No. 65 of Regulation (EU) 2017/745 (MDR).

Contraindications

Relative contraindications:

- Very sensitive scar and skin tissue in the operated area
- Skin diseases
- Radiodermatitis
- Contact dermatitis

Warnings and Precautions

- Follow the instructions for Use
- Follow your surgeon's recommendations regarding use and fitting
- Use the Post-Surgical Bra according to medical indication to avoid injury or damage
- If any unexpected discomfort or wound irritation occurs, remove the Post-Surgical Bra and contact your surgeon
- Make sure that bandages, textile breast prosthesis and/or drainage tubes are placed correctly to avoid pressure points and/or obstructed flow
- Before radiotherapy and X-ray examination remove Post-Surgical Bra except: 3211 Post-Op Bra, Bianca, 3086 Comfort Bra, Catia and 5006/5007 Comfort Bra, Celia
- Wearing an undersized Post-Surgical Bra may cause local pressure pain and disruption of blood flow and/or lymphatic flow
- Wearing an oversized Post-Surgical Bra may reduce the clinical effect

- Use the Post-Surgical Bra according to medical indication to avoid injury or damage.

Performance Characteristics

General Performance Characteristics

- Compression and support zones: Integrated and targeted compression and support zones.
- 4-way stretch: The bra adjusts to the operated breast along with the patient's changing breast contour after surgery.
- Seamless: No seams in critical areas (cups, sides, back and under bust).
- Flexible sizing: Bra fits several sizes due to 4-way stretch and size concept.
- Breathable: Knitted material is breathable.

Specific Performance Characteristics

① 2

Material Composition

Raw materials and components used in the Post-Surgical Bras are tested for harmful substances at minimum Class II per Oeko-Tex Standard 100.

The Post-Surgical Bras are:

- Biological safe, non-toxic; chlorine and dioxin free, and do not contain human or animal tissue or extracts, prohibited azo-dye pigments, medical substances or phthalates
- Free from and unexposed to natural rubber latex or natural rubber derivatives
- Free from metal or other harmful contamination and do not pose any harmful human impact associated with its intended end-use

① 5

PRODUCT	MAIN FABRIC COMPOSITION
3211 Post-Op Bra, Bianca	77% CO, 13% PA, 10% EL
3373 Post-Op Bra, Marianne 3609 Post-Op Bra, Bonnie 3212 Post-Op Bra, Bella 3291 Post-Op Bra, Alice 3270 Post-Op Bra, Anna 3086 Comfort Bra, Catia	91% PA, 9% EL
3342 Post-Op Bra, Sophia	78% CO, 15% PA, 7% EL
3343 Post-Op Bra, Mary	S-2XL: 91% PA, 9% EL 3XL-4XL: 94% PA, 6% EL
3831 Post-Op Bra, Bree 3444 Post-Op Bra, Ava	89% PA, 11% EL
3471 Post-Op Bra, Lisa	55% PA, 33% CO, 9% EL, 3% PES
5006 Comfort Bra, Celia 5007 Comfort Bra, Celia	60% CO, 25% PA, 15% EL

Warranty

- For use-by date, see label on packaging.

Warranty applies within the shelf life period, as long as instruction for use has been complied with.

Measuring Bra Size

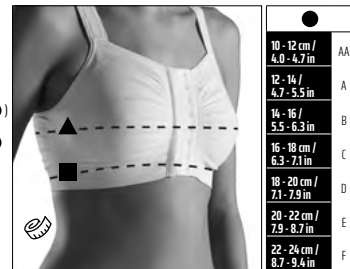
1. Use a standard measuring tape
2. Stand up while measuring
3. Band Size ■: Measure around the upper body, just under your breast
4. Bust Size ▲: Measure around the upper body, at the fullest part of the breast
5. The difference between the Band Size ■ and the Bust Size ▲ shows your Cup Size ● (▲ - ■ = ●)
6. Find your bra size* in the Size Chart printed on the packaging** by using the Band Size ■ and the Cup Size ●

* Due to possible post surgical swelling and edema, you may need to go 1-2 sizes up compared to your normal bra size. Consult your surgeon for guidance.

** The size guide printed on the packaging is unique to each specific Post-Surgical Bra and cannot be used across the bra range.

Instructions for Use

Post-Surgical Bras in primary packaging are prepared for direct clinical use.

- Depending on medical indication, Post-Surgical Bras can be combined with 8113 Implant Stabilizer Band • 8108 Breast Puff can be used with pocketed Post-Surgical Bras

Fitting the bra and accessories

Post-Op Bras

STEPS	PRODUCT(S)	INSTRUCTION
1.	All Post-Op Bras	If medical condition allows, insert one arm and pull the bra over the shoulder. Then insert the other arm. If necessary, ask clinical staff for assistance.
2a.	3373 Post-Op Bra, Marianne 3609 Post-Op Bra, Bonnie 3212 Post-Op Bra, Bella 3291 Post-Op Bra, Alice 3343 Post-Op Bra, Mary 3831 Post-Op Bra, Bree	Pull the under bust band together. Fasten the bottom hook first and work up to the top. Use the widest setting immediately following surgery and the narrowest setting when swelling has subsided.
2b.	3211 Post-Op Bra, Bianca	Pull the under bust band together. Work from the bottom to the top, pressing firmly on hook & loop to close.
2c.	3342 Post-Op Bra, Sophia 3471 Post-Op Bra, Lisa	Pull the middle of the bra together. Fasten the middle hook first, then the other hooks in random order, followed by the zipper.
2d.	3444 Post-Op Bra, Ava 3270 Post-Op Bra, Anna	Pull the bra together. Fasten the top hook first, then the bottom, followed by the zipper.
3.	All Post-Op Bras	Position each breast correctly in the cup. Avoid pulling the bra down too low.
4.	All Post-Op Bras	If the bra has adjustable shoulder straps, adjust these so the bra; does not irritate the skin, provides support under the breasts and sides and feels comfortable to wear.

Comfort Bras

STEPS	PRODUCT(S)	INSTRUCTION
1.	3086 Comfort Bra, Catia 5006 Comfort Bra, Celia 5007 Comfort Bra, Celia	Pull bra over your head or pull it up like underwear.
2.	3086 Comfort Bra, Catia 5006 Comfort Bra, Celia 5007 Comfort Bra, Celia	Adjust the bra and ensure nothing is folded.

Textile Breast Prosthesis

STEPS	PRODUCT(S)	INSTRUCTION
	3373 Post-Op Bra, Marianne 3342 Post-Op Bra, Sophia 3343 Post-Op Bra, Mary 3471 Post-Op Bra, Lisa 3444 Post-Op Bra, Ava	Remove fiberfill from the device until size fits.
		Place the device in the pocket before bra is fitted.

Drainage Bulb

STEPS	PRODUCT(S)	INSTRUCTION
	3373 Post-Op Bra, Marianne	Place drainage bulb in loop and place loop on bra. Press firmly on hook & loop. Max 500 ml.

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3211 Post-Op Bra, Bianca						
3373 Post-Op Bra, Marianne	✓	✓	✓	✓	✓	
3609 Post-Op Bra, Bonnie	✓	✓	✓	✓	✓	✓
3212 Post-Op Bra, Bella	✓	✓	✓	✓	✓	✓
3291 Post-Op Bra, Alice	✓	✓	✓	✓	✓	✓
3270 Post-Op Bra, Anna	✓	✓	✓	✓	✓	✓
3342 Post-Op Bra, Sophia		✓	✓			
3343 Post-Op Bra, Mary	✓	✓	✓	✓	✓	
3831 Post-Op Bra, Bree	✓	✓	✓	✓	✓	✓
3444 Post-Op Bra, Ava	✓	✓	✓	✓	✓	
3471 Post-Op Bra, Lisa		✓	✓			
3086 Comfort Bra, Catia						
5006 Comfort Bra, Celia						✓
5007 Comfort Bra, Celia						✓



EN	MASTECTOMY	LUMPECTOMY	SN BIOPSY AND AXILLARY DISSECTION OF LYMPH NODES	RADIATION TREATMENT
AR	استئصال الثدي جراحياً	استئصال الورم	والنتزيع الإبطي للعدد الليمفاوية SN خزعة الغدة الليمفاوية الحارسة SN	العلاج بالإشعاع
CS	MASTEKTOMIE	LUMPEKTOMIE	BIOPSIE SN A AXILÁRNÍ DISEKCE LYMFATICKÝCH UZLIN	OZÁŘOVÁNÍ
DA	MASTEKTOMI	LUMPEKTOMI	SN-BIOPSI OG FIJERNELSE AF AKSILLÆRE LYMFENUDER	STRÅLEBEHANDLING
DE	MASTEKTOMIE	LUMPEKTOMIE	SN-BIOPSIE UND AXILLADISSEKTION VON LYMPHKNOTEN	BESTRAHLUNG
EL	ΜΑΣΤΕΚΤΟΜΗ	ΟΓΚΕΚΤΟΜΗ	ΒΙΟΨΙΑ SN (ΥΠΕΡΚΛΕΙΔΙΟΥ ΛΕΜΦΑΔΕΝΑ) ΚΑΙ ΜΑΣΧΑΛΙΑΙΑ ΕΚΤΟΜΗ ΤΩΝ ΛΕΜΦΑΔΕΝΩΝ	ΑΚΤΙΝΟΘΕΡΑΠΕΙΑ
ES	MASTECTOMÍA	TUMORECTOMÍA	BIOPSIA GC Y DISECCIÓN AXILAR DE GANGLIOS LINFÁTICOS	RADIOTERAPIA
FI	MASTEKTOMIA	LUMPEKTOMIA	IMUSOLMUKKEEN KOEPAJA JA AKSIAALINEN DISSEKTIO	SÄDEHOITO
FR	MASTECTOMIE	LUMPECTOMIE	BIOPSIE DU GS ET DISSECTION AXILLAIRE DE GANGLIONS LYMPHATIQUES	RADIOTHÉRAPIE
HR	MASTEKTOMIJA	LUMPEKTOMIJA	SN BIOPSIIJA I DISEKCIJA AKSILARNIH LIMFNIH ČVOROVA	LJEČENJE ZRAČENIEM
HU	MASTOPEXIA	LUMPECTOMIA	SN BIOPSIIJA ÉS A NYIROKCSOMÓK AXILLÁRIS METSZÉSE	SUGÁRKEZELÉS
IT	MASTECTOMIA	LUMPECTOMIA	BIOPSIA SN E LINFOADENECTOMIA ASCELLARE	TRATTAMENTO CON RADIAZIONI
JA	乳房切除術	腫瘍切除術	SN生検および腋窩リンパ節郭清	放射線治療
KO	유방절제술	유선종양절제술	SN 생검 및 액와부 림프절 광청술	방사선 치료
NL	MASTECTOMIE	LUMPECTOMIE	SN BIOPSIE EN ONKSELKLIJERDISSECTIE VAN LYMFEEKLIJEREN	BESTRALINGSBEHANDELING
NO	MASTEKTOMI	LUMPEKTOMI	SN-BIOPSI OG AKSILLAR DISSEKSIJON AV LYMFENKJUTER	STRÅLEBEHANDLING
PL	MASTEKTOMIA	LUMPEKTOMIA	BIOPSJA WĘZŁA WARTOWNICZEGO I WYCIĘCIE WĘZŁÓW CHŁONNYCH PACHOWYCH	RADIOTERAPIA
PT	MASTECTOMIA	LUMPECTOMIA	BIÓPSIA DO GÂNGLIO SENTINELA E DISSECÇÃO DE GÂNGLIOS LINFÁTICOS	TRATAMENTOS DE RADIAÇÃO
RU	МАСТЭКТОМИЯ	ЛАМПЭКТОМИЯ	БИОПСИЯ СТОРОЖЕВОГО ЛИМФΟΥЗЛА И ПОДМЫШЕЧНАЯ ЛИМФАДЕНЭКТОМИЯ	ЛУЧЕВАЯ ТЕРАПИЯ
SK	MASTEKTÓMIA	LUMPEKTÓMIA	BIOPSIA SENTINELOVEJ UZLINY A AXILÁRNA DISEKCIJA LYMFATICKÝCH UZLIN	LIEČBA OZÁROVANÍM
SL	MASTEKTOMIJA	LUMPEKTOMIJA	BIOPSIIJA SN IN AKSILARNA DISEKCIJA BEZGAVK	OBSEVANJE
SR	MASTEKTOMIJA	LUMPEKTOMIJA	SN BIOPSIIJA I AKSILARNA DISEKCIJA LIMFNIH ČVOROVA	TRETMAN ZRAČENIEM
SV	MASTEKTOMI	PARTIELL MASTEKTOMI	SN-BIOPSI OCH AXILLÄR DISSEKTION AV LYMFNODER	STRÅLBEHANDLING
CN	乳房切除术	乳房肿瘤切除术	SN 活检和腋窝淋巴结清扫	放射治疗

✓	✓		✓
✓	✓	✓	
✓			
✓			
✓			
✓	✓	✓	
✓	✓	✓	
✓		✓	
✓	✓	✓	
✓	✓	✓	
✓	✓	✓	
✓	✓	✓	
✓	✓	✓	
✓	✓	✓	
✓	✓	✓	

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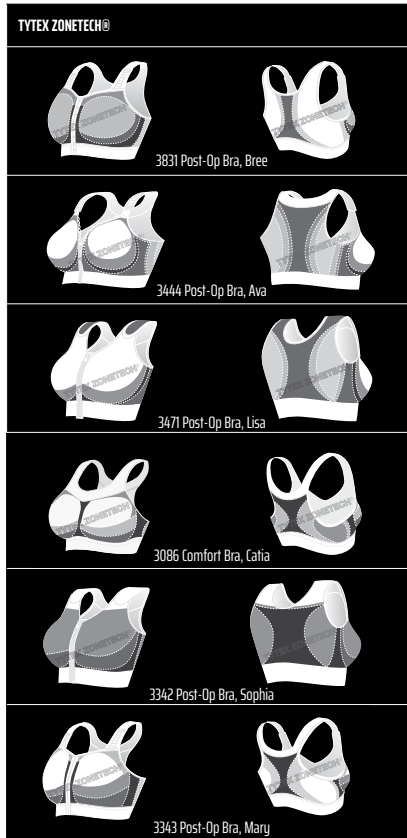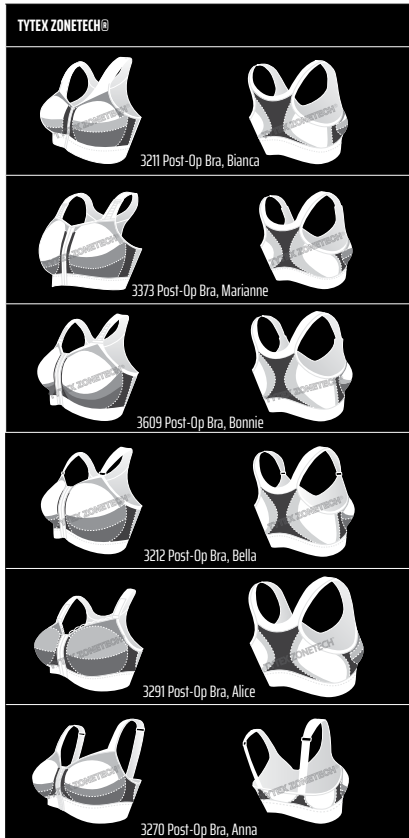carefix[®]

Carefix is a registered trademark of Tytex A/S © 2022.
All rights reserved Tytex A/S, Marsvej 6, DK-7430 Ikast

 Tytex A/S has been certified according to ISO 9001, ISO 14001 and ISO 13485 
TYTEX A/S • Marsvej 6 • DK-7430 Ikast • www.tytex.com

FOLLOW TYTEX