
Clinically proven hip protection

SAFEHIP® ACTIVE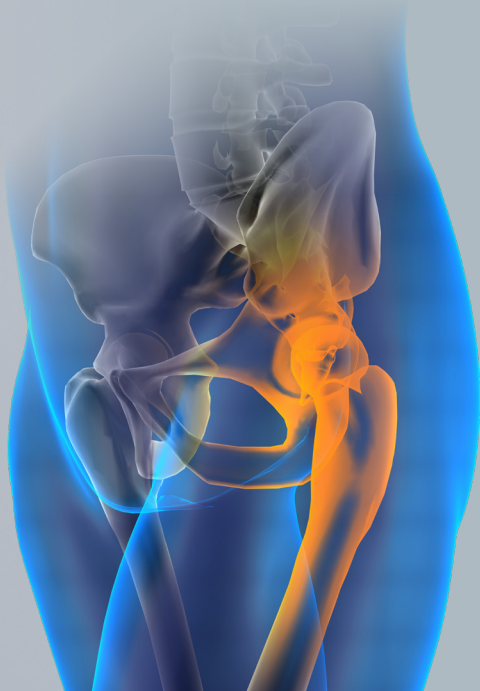

SAFEHIP® THE HIP PROTECTOR

For easy handling, the SAFEHIP® Active belt has hook-&-loop closure and sewn-in protector shields made of lightweight foam. The belt is designed for temporary requirements such as post-surgical mobilisation and training, but it is also useful for active elderly for leisure activities such as walking, running, biking etc.

The SAFEHIP® Active hip protector belt is based on our special horseshoe-shaped shield which not only reduces the energy of the fall impact, but also diverts it into the surrounding soft tissue. SAFEHIP® has been clinically tested on more than 7,000 users around the world and has proven to reduce hip fractures from falls.

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Belt: Spacer fabric
85% Polyester
15% Elastane

Care handling
SAFEHIP® can be washed at 40°C/104°F and tumble dried. Wash with similar colours and in a washing bag. Use of chlorine based bleach or chlorine based disinfection will decrease durability and function.

Please keep this instruction for further reference.


Tytex Customer Service:
E-mail: customerservice@tytex.com
International: +45 9600 4200; +49 2262 762914
Americas: +1 401-762-4100

Tytex A/S has been certified according to ISO 9001, ISO 14001 and ISO 13485
Made in Slovakia by: Tytex A/S · Marsvej 6 · DK-7430 Ikast · info@tytex.dk · www.tytex.com

SAFEHIP®	Cm	Inch	Find your size
S	85-95	34-38	
M	95-105	38-42	
L	105-115	42-46	
XL	115-125	46-50	
XXL	125-140	50-56	

To find the right size, measure the hips at the widest point.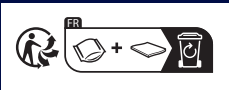

Hip protector · Hüftprotektor · Protecteur de hanches
Heupbeschermer · Hoftebeskytter · Höftskyddsbyxor
Lonkkasuojat · Protector de cadera · Csípővédő

SAFEHIP®
THE HIP PROTECTOR

User Manual

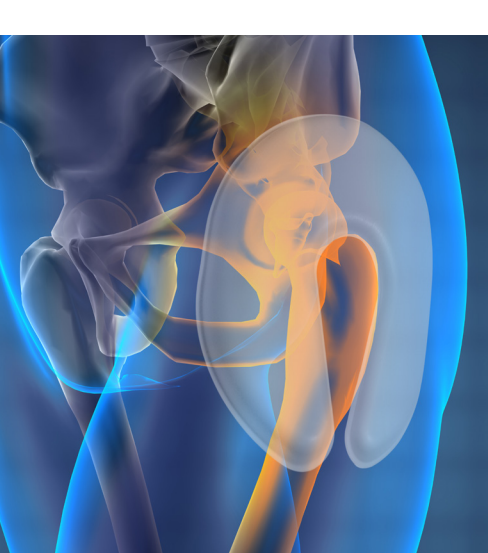
Diagram 1 / Figur 1 / Kuvan 1 / Abb. 1 / Figure 1 / Diagrama 1 / az 1. ábrának / Diagrama 1 / Rysunku 1

Diagram 1 / Figur 1 / Kuvan 1 / Abb. 1 / Figure 1 / Diagrama 1 / az 1. ábrának / Diagrama 1 / Rysunku 1

Diagram 1 / Figur 1 / Kuvan 1 / Abb. 1 / Figure 1 / Diagrama 1 / az 1. ábrának / Diagrama 1 / Rysunku 1

EN Indication
Medical device for prevention of hip fracture in the event of a fall.

Features

SAFEHIP® Active is a hip protector belt with integrated, protective shields. The horseshoe shaped shields protect the critical hip area, exposed to trauma in the event of a fall on the side. The shields both absorb and disperse the energy away from the critical area, providing protection against hip fractures. The hip protector is developed so that the shields will sit in the right position when applying the belt according to the instructions below. The hip protector is made of a spacer fabric that wicks moist away from the body, making the belt very comfortable to wear. SAFEHIP® Active can be used in the daytime, over or under everyday clothing.

Instructions for use

- It is easiest to apply the belt while standing.
- Make sure that the belt is turned correctly. It is very important that the open end of the horseshoe shaped shields is facing downward towards the feet.
- Position SAFEHIP® Active so that both shields are correctly centered over the critical area, the hip. The Safehir shields provide a protective ring around the bony prominence as shown in diagram 1.
- Close the hook-&-loop fastening in the front and adjust for comfort and fit. The belt should be adjusted so that it does not shift or turn around the body. Do not tighten excessively so it impairs your comfort.
- Please check that the SAFEHIP® Active and especially the shields are positioned as shown in Diagram 1.
- At regular intervals during daily activities, it is recommended to check that SAFEHIP® Active is positioned correctly according to these instructions.
- If in doubt about appliance or fitting, please consult a doctor or nurse to guide you in fitting the belt correctly.

Precautions

No hip protector can be 100% effective in preventing injury from falls, and wearing SAFEHIP® does not guarantee protection from injury. It is important always to ensure correct shield positioning, every time SAFEHIP® is applied. SAFEHIP® has been designed to resist multiple falls. We recommend that you check for any irregularities during the lifetime of the product and after each fall. If any damage occurs, the product must not be worn. Any persistent indentation is to be considered damage. If SAFEHIP® Active is used in sports activities where the energy in falls may be considerably greater than that of daily life activities, the protection offered by the belt may be reduced and in worst case insufficient to reduce the fracture risk. SAFEHIP® is only to be used for its intended use of hip protection - it is not to be used in conjunction with other medical devices without consulting a doctor first. If any unexpected discomfort or skin irritation occurs, please consult your doctor or nurse.

Care Handling

SAFEHIP® can be washed at 40°C/104°F and tumble dried. Close hook-&-loop before washing. Wash SAFEHIP® Active before using it for the first time. Wash with similar colours and in a washing bag. Use of chlorine based bleach, chlorine based disinfection or washing powder will decrease durability and function. If it is absolutely necessary to disinfect SAFEHIP® Active, we recommend using Virkon.

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