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Clinically proven and breathable hip protection

SAFEHIP® AirX®

SAFEHIP® THE HIP PROTECTOR

The protective shields in SAFEHIP® AirX® are 100% textile, made of AirX® spacer fabric technology. This ensures a breathable, skin friendly and very soft hip protector, preventing moisture build-up so that the product will always feel comfortable against your skin. The great advantages of an AirX® textile hip protector are that the shields conform perfectly to the body shape and feel cool to wear, and that the product can be washed and tumble dried at very high temperatures for high hygiene.

The shields are based on the special horseshoe-shape, which not only reduces the energy of the impact of the fall, but also diverts it into the surrounding soft tissue. SAFEHIP® has been clinically tested on more than 7,000 users around the world and has proven to reduce hip fractures from falls.

AirX®
100%
BREATHABLE
100%
TEXTILE

Protector
100% AirX® soft textile

Seamless Briefs

58% Cotton
36% Polyamide
6% Elastane

Care handling

SAFEHIP® can be washed at 95°C/200°F and tumble dried. Wash with similar colours and in a washing bag. Use of chlorine based bleach or chlorine based disinfection will decrease durability and function.

Please keep this instruction for further reference.

Tytex Customer Service:
E-mail: customerservice@tytex.com
International: +45 9660 4200; +49 2262 762914
Americas: +1 401-762-4100

SAFEHIP®	Cm	Inch	Find your size
XS	65-85	26-34	
S	75-95	30-38	
M	90-110	36-44	
L	100-120	40-48	
XL	110-140	44-56	
XXL	120-150	48-60	

Sizes overlap. To find the user's correct size, measure the hips at the widest point. If the hip measurement lies between two sizes, choose the smaller size if the patient has slim thighs, and the larger size if thighs are wide or an incontinence pad will be worn.

Hip protector · Hüftprotector
Protecteur de hanches · Heupbeschermer
Hoftebeskytter · Höftskyddsbyxor · Lonkkasuojat
Protector de cadera · Protezione dell'anca · Csípővédő

SAFEHIP® THE HIP PROTECTOR

Tytex A/S has been certified according to ISO 9001, ISO 14001 and ISO 13485
Made in Slovakia by: Tytex A/S · Marsvej 6 · DK-7430 Ikast · info@tytex.dk · www.tytex.com

TYTEX®
MAKING HEALTHCARE MORE HUMAN

User Manual

Diagram 1 / Figur 1 / Kuvan 1 / Abb. 1 / Figure 1 / Diagrama 1 / az 1. ábrának / Diagrama 1 / Rysunku 1

Diagram 1 / Figur 1 / Kuvan 1 / Abb. 1 / Figure 1 / Diagrama 1 / az 1. ábrának / Diagrama 1 / Rysunku 1

Diagram 1 / Figur 1 / Kuvan 1 / Abb. 1 / Figure 1 / Diagrama 1 / az 1. ábrának / Diagrama 1 / Rysunku 1

EN
SAFEHIP® provides clinically proven and economic solutions to prevent hip fracture for individuals who have fallen or are at risk of falling. Clinical trials have proven that the use of SAFEHIP® reduces the risk of a hip fracture. SAFEHIP® provides protection against hip fractures when both SAFEHIP® shields are placed correctly in both pockets of the SAFEHIP® briefs. However, no hip protector can be 100% effective in preventing injury from all falls and wearing SAFEHIP® does not guarantee protection from injury. SAFEHIP® briefs are developed so that shields will sit in the right position. It is important that both shields are correctly centered over the greater trochanter as shown in Diagram 1.

General disclaimer

SAFEHIP® has been designed to resist multiple falls. We recommend that you check for any irregularities during the lifetime of the product and after each fall. If any damage occurs the product must not be worn. Any tear, scratch or persistent indentation is to be considered damage. When using SAFEHIP® it is important to check the area of the body in direct contact with the product for signs of skin deterioration. If you see any examples of skin deterioration please seek medical advice. SAFEHIP® is only to be used for its intended use of hip protection and incontinence pad fixation - it is not to be used in conjunction with other medical devices without first consulting a doctor.

Care Instructions

We recommend following the care instructions: Safehir® can be washed at up to 95°C (200°F) and tumble dried. Best durability will be achieved by washing at 60°C (140°F). It is recommended to use a washing bag. Do not use bleach as this will damage the SAFEHIP® over time. Bleach powder should not be used under any circumstances.

Important: Industrial laundering

In general, the washing instructions in the SAFEHIP® briefs should be followed. In connection with industrial laundering SAFEHIP® should never be

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