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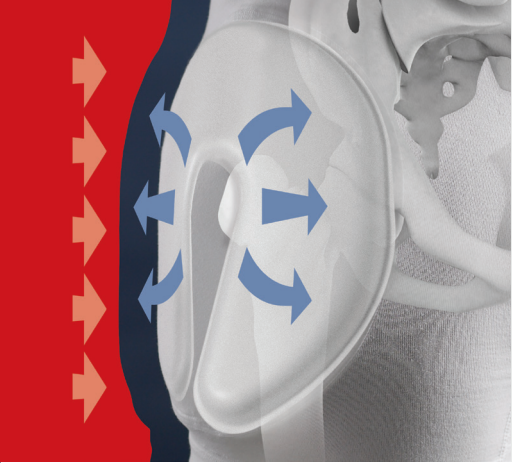
Clinically proven hip protection

SAFEHIP®
THE HIP PROTECTOR

The protective shields in SAFEHIP® Classic are based on our specially horseshoe-shaped foam shield. The shields are light-weight, shaped to fit the body's contour and feel very comfortable to wear. The pants hold a high content of elastane which ensures a perfect fit and also makes them suitable for fixation of an incontinence pad.

The patented shield technology of SAFEHIP® uses an energy shunting technique to reduce the fall impact by dispersing the energy of the fall away from the hip area and into the soft tissue. SAFEHIP® has been clinically tested on more than 7,000 users around the world and has proven to reduce hip fractures from falls.

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Seamless Briefs

67% Cotton
29% Polyester
4% Elastane

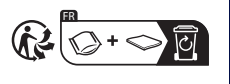
Care handling

SAFEHIP® can be washed at 60°C/140°F and tumble dried. Wash with similar colours and in a washing bag. Use of chlorine based bleach or chlorine based disinfection will decrease durability and function.

Please keep this instruction for further reference.

Tytex Customer Service:
E-mail: customerservice@tytex.com
International: +45 9660 4200; +49 2262 762914
Americas: +1 401-762-4100

SAFEHIP®	Cm	Inch	Find your size
XS	65-85	26-34	
S	75-95	30-38	
M	90-110	36-44	
L	100-120	40-48	
XL	110-140	44-56	
XXL	120-150	48-60	

Sizes overlap. To find the user's correct size, measure the hips at the widest point. If the hip measurement lies between two sizes, choose the smaller size if the patient has slim thighs, and the larger size if thighs are wide or an incontinence pad will be worn.

 Tytex A/S has been certified according to ISO 9001, ISO 14001 and ISO 13485
Made in Slovakia by: Tytex A/S · Marsvej 6 · DK-7430 Ikast · info@tytex.dk · www.tytex.com

TYTEX®
MAKING HEALTHCARE MORE HUMAN

User Manual

Diagram 1 / Figur 1 / Kuvan 1 / Abb. 1 / Figure 1 / Diagrama 1 / az 1. ábrának / Diagrama 1 / Rysunku 1

EN

SAFEHIP® provides clinically proven and economic solutions to prevent hip fracture for individuals who have fallen or are at risk of falling. Clinical trials have proven that the use of SAFEHIP® reduces the risk of a hip fracture. SAFEHIP® provides protection against hip fractures when both SAFEHIP® shields are placed correctly in both pockets of the SAFEHIP® briefs. However, no hip protector can be 100% effective in preventing injury from all falls and wearing SAFEHIP® does not guarantee protection from injury. SAFEHIP® briefs are developed so that shields will sit in the right position. It is important that both shields are correctly centered over the greater trochanter as shown in Diagram 1.

General disclaimer

SAFEHIP® has been designed to resist multiple falls. We recommend that you check for any irregularities during the lifetime of the product and after each fall. If any damage occurs the product must not be worn. Any tear, scratch or persistent indentation is to be considered damage. When using SAFEHIP® it is important to check the area of the body in direct contact with the product for signs of skin deterioration. If you see any examples of skin deterioration please seek medical advice. SAFEHIP® is only to be used for its intended use of hip protection and incontinence pad fixation - it is not to be used in conjunction with other medical devices without first consulting a doctor.

Care Instructions

We recommend following the care instructions: SAFEHIP® can be washed at up to maximum 60° C / 140 °F. Washing at a higher temperature will deform and/or shrink the shields, and if this happens, use must be discontinued. It is recommended to use a washing bag. Do not use bleach as this will damage the SAFEHIP® over time. Bleach powder should not be used under any circumstances. When drying in a dry cabinet, the product must not touch the heating unit.

Important: Industrial laundering
In general, the washing instructions in the SAFEHIP® briefs should be followed. In connection with industrial laundering SAFEHIP® should never be subjected to

excessive pressure during the washing or drying process as pressure combined with heat may deform the shields.

Safehip® with removable shields

Removable shields should be placed in the briefs before using. The shields must be placed in the pockets, with the legs of the horseshoe contour pointing down. Safehip® shields should only be used in combination with Safehip® briefs.

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